

Chemotherapy Spill Clean-Up Kit



Safetec of America, Inc.
Buffalo, NY 14215
1-800-456-7077
www.safetec.com



Directions for use:



1. Put on disposable Gloves, Gown, Shoe Covers and Safety Shield.



2. Sprinkle Green-Z[®] over spill evenly and allow liquid to congeal for safe handling.



3. Remove collected material with Scoop and Scraper and place in Yellow Chemotherapy Waste Bag.



4. Wipe up any residue left behind with supplied Wiper Pads. Clean area with a hard surface wipe or disinfectant.



5. Remove protective clothing and place all contaminated materials (Wiper Pads, Gloves, Gown, Shoe Covers and Shield) in supplied Yellow Chemotherapy Waste Bag. Seal bag with supplied Twist Tie.

6. Fill out Identification Tag and attach. Dispose of bag in accordance with local, state, and federal regulations.

Contents:

2 pair	Nitrile Chemotherapy Gloves
1 each	Chemotherapy Gown
1 pair	Shoe Covers, non-slip
1 each	full-face Safety Shield
1 each	21gram Green-Z [®] Fluid Solidifier
1 each	Scoop & Scraper
1 each	Yellow Chemotherapy Waste Bag
2 each	Wiper Pads
1 each	Twist Tie
1 each	Identification Tag

REORDER NO.
49725

CAUTION!



CHEMOTHERAPY

DRUG SPILL

Proceed With Care!
See reverse side for Directions

